

Instructions for use



GTIN	Product	Description
8908019116106	CLEANCARE - Operation Theatre Shoe Cover	Sterile Operation Theatre Shoe Cover

Indications for use

To preserve a sterile or clean zone such as an operative / intraoperative environment, the healthcare practitioner should wear the Operation Theatre Shoe Cover on their footwear.

Instructions for use

1. Take out Shoe Covers one by one from the sachet and carefully unfold.
2. Wear the Shoe Cover over the footwear, this must be done in a sterile area.
3. Ensure that the Shoe Cover is comfortably covering the footwear and should extend above the bottom hem of the trousers or up to the lower calf.
4. Secure the Shoe Cover with the provided self-adhesive sticker.
5. After use, dispose the product in accordance with your facility's internal guidelines.

Storage Conditions

Store in a cool dry place.