

Instructions for use



GTIN	Product	Description
8908019116168	Drape Sheet	Patient undercovering

Indications for use

Hi Care Remedy's, Sterilised Drape Sheets are intended to protect the interface between the hospital bed mattress and the patient. The Drape sheet should be placed under the hospital bedsheet.

The Drape Sheet is sterile, for single patient use and disposable. The drape sheet should be replaced whenever there is accumulation of patient fluids.

Instructions for use

1. Take out the Drape sheet from the sachet. Unfold the drape sheet carefully.
2. Lay the drape sheet on the bed mattress, patting down to avoid surface wrinkles and undulations.
3. Secure the Drape sheet at the four corners using the attached self-adhesive stickers. For added hold use the two loose self-adhesive stickers as required.
4. After use, dispose the product in accordance with your facility's internal guidelines.

Storage Conditions

Store in a cool dry place.