

Instructions for use (IFU)

Rev 1 / March 2024

Download the latest IFU from www.hicareindia.com

GTIN	Product	Description
8908019116090	CLEARPROBE™	Endocavity / TVS Probe cover

Indications for use

CLEARPROBE™ Probe Covers may be used as a protective cover over Endocavity ultrasound transducer/probe - to protect patients, healthcare practitioners, and the ultrasound equipment from cross-contamination.

CLEARPROBE™ Probe Covers are sterile, for single patient/procedure use and disposable.

Instructions for use

1. Apply gel to the Endocavity probe/transducer head prior to inserting the Endocavity probe/transducer head into the cover.
2. Pull the probe cover all the way down and secure the to the probe using the attached self-adhesive sticker.
3. Inspect probe cover to ensure there are no holes or tears.
4. After use, dispose the product in accordance with your facility's internal guidelines.

Storage Conditions

Store in a cool dry place.



Manufacturer



Batch code/ Lot number



Date of Manufacture



Use by date



Do not re-use/ Single use only



Consult instructions for use or consult electronic instructions for use



Medical Device



Latex Free



Do not use if package is damaged



Sterilised using ethylene oxide



Keep dry



Keep away from sunlight