

# Instructions for use



GTIN	Product	Description
8908019116304	Sterilised Camera Cover	Sterilised Camera Cover

## Indications for use

Hi Care Remedy's, Sterilised Camera Cover / Camera Sleeve is intended to provide a sterile interface between the operation theatre camera and the healthcare professional. The Camera Cover should be used by trained healthcare professionals.

The Camera Cover is sterile, for single patient/procedure use and disposable.

## Instructions for use

1. Pull the Camera Cover / Camera Sleeve so that it covers the cable. Ensure that the lens is not obstructed. Secure the Camera Cover using the provided self-adhesive sticker and elastic bands.
2. Inspect the Camera Cover to ensure there are no holes or tears.
3. After use, dispose the product in accordance with your facility's internal guidelines.

## Storage Conditions

Store in a cool dry place.