

Instructions for use (IFU)

Rev 1 / March 2024

Download the latest IFU from www.hicareindia.com

GTIN	Product	Description
8908019116434	Ultrasound Probe Cover	General Purpose Ultrasound Probe Cover (Flat folded) 12" x 7" (305 x178mm)
8908019116434	Ultrasound Probe Cover	General Purpose Ultrasound Probe Cover (Flat folded) 24" x 7" (610 x178mm)
8908019116434	Ultrasound Probe Cover	General Purpose Ultrasound Probe Cover (Flat folded) 48" x 7" (1220 x178mm)
8908019116434	Ultrasound Probe Cover	General Purpose Ultrasound Probe Cover (Flat folded) 59" x 7" (1500 x178mm)

Indications for use

Hi-Care Remedy's General Purpose Ultrasound Probe Cover, maybe used as a protective cover over medical ultrasound transducer/probe to protect patients, healthcare practitioners, and the ultrasound equipment from cross-contamination.


General Purpose Ultrasound Probe Cover are sterile, for single patient/procedure use and disposable.


Instructions for use


1. Apply gel to the probe/transducer head prior to inserting the probe/transducer head into the cover.
2. Pull the probe cover all the way down and secure to the probe and cable using the provided self-adhesive sticker and elastic bands.
3. Inspect probe cover to ensure there are no holes or tears.
4. After use, dispose the product in accordance with your facility's internal guidelines.


Storage Conditions


Store in a cool dry place.


 Manufacturer


 Batch code/ Lot number


 Date of Manufacture


 Use by date


 Do not re-use/ Single use only


 Consult instructions for use or consult electronic instructions for use


 Medical Device

 Latex Free

 Do not use if package is damaged

 Sterilised using ethylene oxide

 Keep dry

 Keep away from sunlight