

# Instructions for use (IFU)

Rev 1 / March 2024

Download the latest IFU from [www.hicareindia.com](http://www.hicareindia.com)

GTIN	Product	Description
8908019116021	HANDCARE™ Non-Sterile (Large)	EVA Non-Sterile Gloves Large (L) 8-8.5
8908019116045	HANDCARE™ Non-Sterile (Medium)	EVA Non-Sterile Gloves Medium (M) 7-7.5
8908019116069	HANDCARE™ Non-Sterile (Small)	EVA Non-Sterile Gloves Small (S) 6-6.5
	HANDCARE™ Non-Sterile (Extra Small)	EVA Non-Sterile Gloves Extra Small (XS) 5-5.5

## Indications for use












HANDCARE™ non-sterile gloves can be used for patient care but is not recommended for sterile procedures. Additionally, HANDCARE™ non-sterile gloves can be used during hospitality, food handling, beauty applications.

## Instructions for use

1. Remove any objects from your hands.
2. Ensure hands are clean.
3. Take out gloves one by one from the carton.
4. The gloves are ambidextrous, hence can be worn on either hand.
5. Avoid excess force when donning the glove.
6. After use, dispose the product in accordance with your facility's internal guidelines.

## Storage Conditions

Store in a cool dry place.

-  Manufacturer
-  Batch code/ Lot number
-  Date of Manufacture
-  Use by date
-  Do not re-use/ Single use only
-  Consult instructions for use or consult electronic instructions for use
-  Medical Device
-  Latex Free
-  Do not use if package is damaged
-  Keep dry
-  Keep away from sunlight