

# Instructions for use (IFU)

Rev 1 / March 2024

Download the latest IFU from [www.hicareindia.com](http://www.hicareindia.com)

GTIN	Product	Description
8908019116366	HANDCARE™ Sterile (Large)	EVA Sterile Gloves Large (L) 8-8.5
8908019116342	HANDCARE™ Sterile (Medium)	EVA Sterile Gloves Medium (M) 7-7.5
8908019116359	HANDCARE™ Sterile (Small)	EVA Sterile Gloves Small (S) 6-6.5
8908019116335	HANDCARE™ Sterile (Extra Small)	EVA Mini Gloves Extra Small (XS) 5-5.5

## Indications for use











HANDCARE™ Sterile gloves should be used before a sterile procedure, when anticipating contact with blood or other body fluids, when making contact with the patient or the patient's immediate surroundings.

## Instructions for use

1. Remove any objects from your hands.
2. Ensure hands are clean.
3. Take out gloves one by one from the sachet.
4. The gloves are ambidextrous, hence can be worn on either hand.
5. Avoid excess force when donning the glove.
6. After use, dispose the product in accordance with your facility's internal guidelines.

## Storage Conditions

Store in a cool dry place.

-  Manufacturer
-  Batch code/ Lot number
-  Date of Manufacture
-  Use by date
-  Do not re-use/ Single use only
-  Consult instructions for use or consult electronic instructions for use
-  Medical Device
-  Latex Free
-  Do not use if package is damaged
-  Sterilized using ethylene oxide
-  Keep dry
-  Keep away from sunlight