

# Instructions for use (IFU)

Rev 1 / March 2024

Download the latest IFU from [www.hicareindia.com](http://www.hicareindia.com)

GTIN	Product	Description
8908019116403	HANDCARE™ Sterile with Wristband (Large)	EVA Sterile Gloves with Wristband Large (L) 8-8.5
8908019116397	HANDCARE™ Sterile with Wristband (Medium)	EVA Sterile Gloves with Wristband Medium (M) 7-7.5
8908019116380	HANDCARE™ Sterile with Wristband (Small)	EVA Sterile Gloves with Wristband Small (S) 6-6.5
8908019116373	HANDCARE™ Sterile with Wristband (Extra Small)	EVA Sterile Gloves with Wristband Extra Small (XS) 5-5.5

## Indications for use

HANDCARE™ Sterile gloves with Wristband should be used before a sterile procedure, when anticipating contact with blood or other body fluids, when making contact with the patient or the patient's immediate surroundings.


## Instructions for use


1. Remove any objects from your hands.
2. Ensure hands are clean.
3. Take out gloves one by one from the sachet.
4. The gloves are ambidextrous, hence can be worn on either hand.
5. Avoid excess force when donning the glove.
6. Secure the self-adhesive band at the wrist.
7. After use, dispose the product in accordance with your facility's internal guidelines.


## Storage Conditions


Store in a cool dry place.


 Manufacturer


 Batch code/ Lot number


 Date of Manufacture


 Use by date


 Do not re-use/ Single use only


 Consult instructions for use or consult electronic instructions for use


 Medical Device

 Latex Free

 Do not use if package is damaged

 Sterilised using ethylene oxide

 Keep dry

 Keep away from sunlight